



FITTING PROCEDURE – STEPS 1 TO 5


1. THESE ARE YOUR PERSONALLY MOULDED EARPLUGS. THE RIGHT EARPLUG HAS A RED DOT, THE LEFT IS NOT MARKED

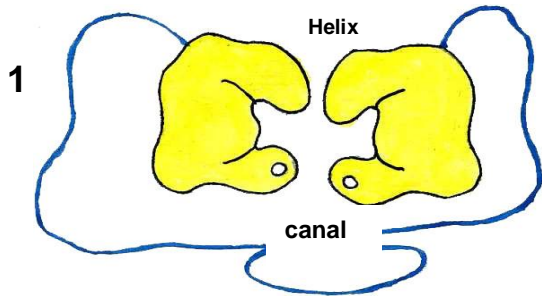


2. TO FIT LEFT EARPLUG USE CLEAN HANDS & HOLD EARPLUG WITH INDEX FINGER & THUMB WITH FLAT SIDE FACING AWAY FROM THE EAR.

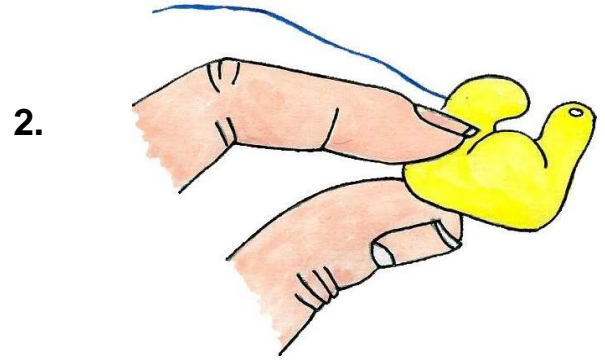


3. PLACE EARPLUG CANAL INTO THE EAR CANAL GENTLY TWIST BACKWARDS UNTIL THE EARPLUG SLIDES COMFORTABLY INTO EAR BOWL.

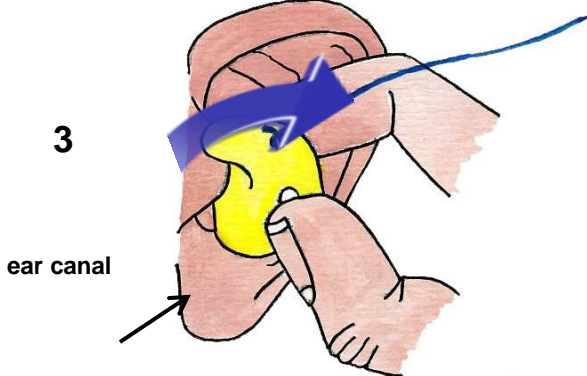

4. PLACE EARPLUG HELIX INTO THE HELIX OF THE EAR.

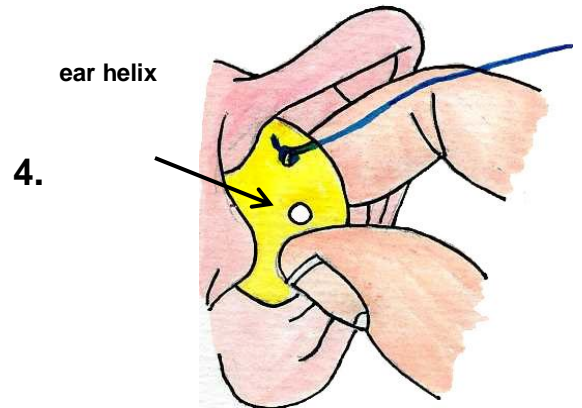


5. TO ENSURE CORRECT FIT, PUSH GENTLY ON FLAT AREA TO SEAT EARPLUG INTO BOWL OF EAR. REPEAT THE PROCEDURE FOR RIGHT EAR.

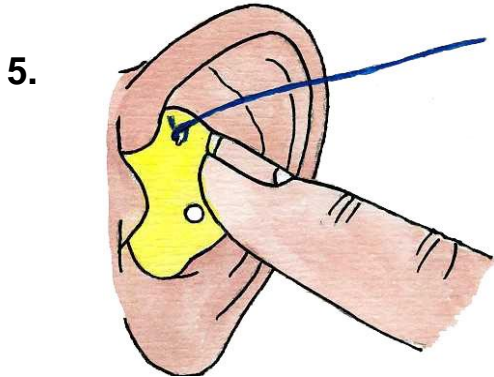


REMOVAL PROCEDURE – STEPS 6 TO 8


6. TO REMOVE LEFT EARPLUG WITH CLEAN HANDS USE INDEX FINGER TO PRESS ON THE BACK OF EARFLAP TO LIFT EARPLUG AWAY FROM EAR BOWL.

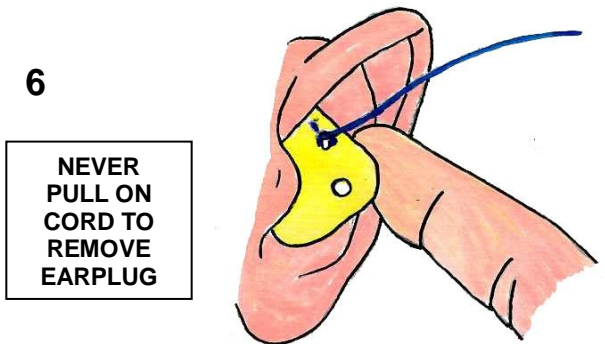

7. SLIDE INDEX FINGER BEHIND EARPLUG & GRIP FRONT WITH THUMB.

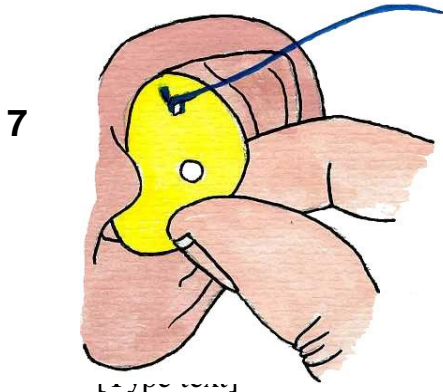

8. CAREFULLY REMOVE EARPLUG WITH INDEX FINGER & THUMB BY GENTLY TWISTING FORWARDS

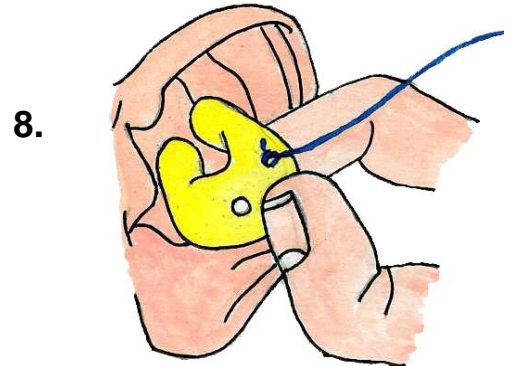

FOR A VIDEO DEMONSTRATION, SEARCH 'PROTECTHEAR' ON